



**Re: INTERCEPT Blood System™ for Platelets and Plasma
Product Code INT100: Expanded Disinfectant List for the INTERCEPT Illuminator**

Dear Valued Customer:

In response to your feedback, we are pleased to advise you of an expansion to the approved labeling for the INTERCEPT Illuminator.

INTERCEPT Illuminator Disinfectant List Expanded to Include Ten New Disinfectants

Ten new disinfectants have been added to the list of products suitable for use with the INTERCEPT Illuminator. Table 1 specifies the disinfectants tested. Per the table, please note that within the set of new disinfectants, Bacillol AF should **NOT** be used on the illumination tray. Use a soft cloth to apply the disinfectants. After the manufacturer’s recommended contact time, use a soft cloth dampened with water to remove the solution. All other preparation/application parameters should be as per the disinfectant manufacturer’s instructions. Disinfectants were deemed suitable based on material compatibility with illuminator surfaces; Cerus makes no claims about the microbial disinfection efficacy of the products.

Table 1 Disinfectants Suitable for Use with INTERCEPT Illuminators

Disinfectant Name and Manufacturer	Suitable for Use?			
	Illumination Tray	Reflective Divider in Tray	Screen and Keypad	External Surface
Freshly Prepared Bleach (10%) (Sodium hypochlorite)	YES	NO	YES	YES
Iodine based disinfectant	YES	YES	NO	YES
LpH se disinfectant	NO	YES	YES	YES
NEW				
Bacillol AF (ready to use) (Bode, Germany)	NO	YES	YES	YES
CaviCide (ready to use) (Metrex, USA)	YES	YES	YES	YES
DAX 70+ (ready to use) (Opus Health Care, Sweden)	YES	YES	YES	YES
Gigasept FF New (5%) (Schulke & Meyr, Germany)	YES	YES	YES	YES
Incidin Plus (2%) (Ecolab, Germany)	YES	YES	YES	YES
Kohrsolin Extra (0.5%) (Bode, Germany)	YES	YES	YES	YES
Mikrobac Tissues (ready to use) (Bode, Germany)	YES	YES	YES	YES
Mikrozid Sensitive Liquid (ready to use) (Schulke & Meyr, Germany)	YES	YES	YES	YES
M-Ytides (ready to use) (Kemetyl, Sweden)	YES	YES	YES	YES
Terralin Protect (2%) (Schulke & Meyr, Germany)	YES	YES	YES	YES



Implementation

Adoption of new disinfectants should proceed according to the internal processes of your blood establishment and in compliance with local regulations and procedures.

Contact information

If you have any questions concerning this change, please contact Cerus BV Customer Services:

Stationsstraat 79-D
3811 MH Amersfoort
The Netherlands

Email: customer_services@cerus.com
Phone: +31 33 496 0600
Fax: +31 33 496 0606

With Kind Regards,

Cerus Europe BV